

KERN

KERN & Sohn GmbH

Ziegelei 1
D-72336 Balingen
E-Mail: info@kern-sohn.com

Phone: +49-[0]7433- 9933-0
Fax: +49-[0]7433-9933-149
Internet: www.kern-sohn.com

Assembly instructions Foot stopper, Head stopper

KERN MBC-A05

Version 1.0
04/2014
GB



MBC-A05-IA-e-1410



KERN MBC-A05

Version 1.0 04/2014

Assembly instructions foot stopper, head stopper

Contents

1	Technical data.....	3
2	Scope of delivery	3
3	Description	3
4	Assembly	4
5	Cleaning/Disinfect	4

English

1 Technical data

KERN	MBC-A05
Dimensions (mm), completely assembled	600 x 280 x 128
Net weight (g)	350

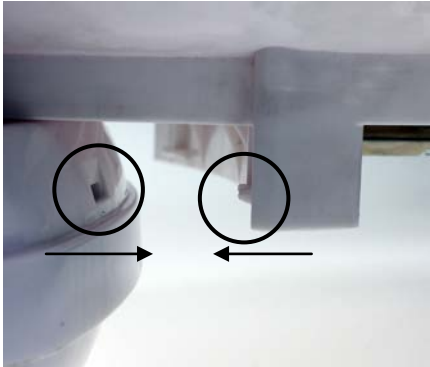

2 Scope of delivery

- 1 Weighing plate with foot stopper and head stopper
- Assembly instructions

3 Description

The KERN MBC-A05 foot and head stopper limits the weighing plate by the side, increasing by that way your baby's safety.

4 Assembly

	<p>For assembly fix the foot and head stopper on balance acc. to illustration.</p>
	<p>Foot and head stopper assembled.</p>

5 Cleaning/Disinfect

Weighing plate and display unit scale should be cleaned exclusively with household cleaners or commercially available disinfectants. Follow the manufacturer's recommendations.

Do not use any polishing or aggressive cleaning agents (white spirit, petrol etc.) as they may damage the high-quality surface.

To avoid cross contamination (mycosis, ...), the scale plate is to be cleaned regularly. Recommendation: after each weighing which could result in potential contamination (e.g. when there is a direct skin contact during weighing).



Do not spray disinfectants onto appliance.

Make sure that disinfectant does not penetrate the interior of the appliance.

We recommend wiping disinfection.

Remove dirt immediately.